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Becton Dickinson Pty Ltd., 66 Waterloo Road, Macquarie Park NSW 2113, Australia

Becton Dickinson Limited, 14B George Bourke Drive, Mt. Wellington Auckland 1060, Nowa Zelandia

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BD Vacutainer® PrecisionGlide™ Multiple Sample Blood Collection Needle

REF	360220	REF	360224
REF	360221	REF	360225
REF	360222	REF	360226
REF	360223	REF	360227

CE
0123

MD

STERILE EO

SPECIFICATION

- OD of the Needle:** 0.5 mm (25G), 0.55 mm (24G), 0.6 mm (23G), 0.7 mm (22G), 0.8 mm (21G), 0.9 mm (20G), 1.1 mm (19G), 1.2 mm (18G), 1.4mm(17G), 1.6mm(16G);
- Needle length:** 25 mm ~ 40 mm;
- Example:** The OD of the Needle is 21G, Needle length is 25 mm, will be marked as **21Gx1"(0.8x25mm)**;

22Gx1" (0.7x25mm)	22Gx1 1/2" (0.7x38mm)	21Gx1" (0.8x25mm)	21Gx1 1/2" (0.8x38mm)	20Gx1" (0.9x25mm)	20Gx1 1/2" (0.9x38mm)	18Gx1" (1.2x25mm)	18Gx1 1/2" (1.2x38mm)
360220	360221	360222	360223	360224	360225	360226	360227

INTENDED PURPOSE

Multiple Sample Blood Collection Needle is intended for medical purpose to use with Vacuum tubes for venous blood specimen for venous puncture to collect blood samples of patients. It can only be used once and shall be disposed as medical waste after use.

INTENDED USERS

Healthcare personnel with Multiple Sample Blood Collection Needle experience.

CONTRAINDICATIONS

None

CAUTIONS AND WARNING

- Please check the package of this product before use; Do not use if package is damaged.
- Pay attention to check whether the Needle cap dropped off, if any, don't use the product.
- This product is only for single use and should not be reused.
- Use it immediately after opening the primary package and disposed as medical waste after use;
- Use the product before the expiry date marked on the label.
- During operation, the user should be careful not to be needlestick.
- Make sure that the Needle tip is not damaged or any burrs before use. If so, do not use it.
- This product should be stored in a cool, dry, clean, well ventilated environment without corrosive gas.
- This device contains the following substance defined as CMR 1B, in a concentration above 0.1% weight by weight Cobalt, CAS Number 7440-48-4 and EC Number 231-158-0. Current scientific evidence supports that medical devices manufactured from cobalt alloys or stainless steel alloys containing cobalt do not cause an increased risk of cancer or adverse reproductive effect.

Note: For more information, please consult the European Chemicals Agency website: echa.europa.eu/home.

POTENTIAL ADVERSE REACTION

- Vascular injuries;
- Redness.

HOW TO USE

- Check whether the Label is damaged; whether the Cap is damaged or fell off;
- Rotate the Needle Cap and End Cap to break the label, and remove the Cap;
- Insert the Puncturing Needle End, and screw in the Luer Fitting of Needle holder;
- Remove the Needle Cap, perform the venous puncturing immediately;
- Insert the Blood Collection Needle into the Needle holder, and align the Puncturing Needle End for puncturing, then to collect blood sample;

STERILIZATION EO Sterilization.

SHELF LIFE 5 years.

STORAGE AND TRANSPORTATION CONDITIONS

The product should be stored in a cool, dry and clean room and shall be protected from heavy pressure and rain during transportation.

Note: If any serious incident that has occurred in relation to the device, please report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

EUROPEAN REPRESENTATIVE

EU REP Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80, 20537 Hamburg, Germany

Becton Dickinson Distribution Center NV, Laagstraat 57, 9140 Temse, Belgium
Becton Dickinson AG, Binningerstrasse 94, 4123 Allschwil, Switzerland

CH REP

SUNGO Technical Service GmbH. Bahnhofstrasse 21, 6300 Zug, Switzerland
Australian and New Zealand Sponsors:

Becton Dickinson Pty Ltd., 66 Waterloo Road, Macquarie Park NSW 2113, Australia
Becton Dickinson Limited, 14B George Bourke Drive, Mt. Wellington Auckland 1060, New Zealand

BD, the BD Logo, PrecisionGlide, and Vacutainer are trademarks of Becton, Dickinson and Company or its affiliates. © 2024 BD. All rights reserved. Made in China

MANUFACTURER

Berpu Medical Technology Co., Ltd.

No.14 Xingji Road, Yongxing Street, Longwan District, 325000 Wenzhou,
Zhejiang Province, PEOPLE'S REPUBLIC OF CHINA