Français

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English

INSTRUCTIONS FOR USE

MICRO CANNULAS AND KITS FOR FILLERS INJECTION

Description

The Aesthetic Group Micro Cannula is a disposable device exclusively dedicated to the injection of a filler (synthetic or autologous fat) under the skin for aesthetic body contouring. Packaged sterile.

The blunt tip allows atraumatic introduction, reducing the risk of hematoma and vessel or tissues injury.

The cannulas are made of high-quality steel and have thin walls with a smooth surface, which makes them easy to insert and without trauma.

The cannula hub has an ergonomic shape for easier handling when attached to or removed from the syringe. It also has a color code to indicate the diameter of the cannula.

It is equipped with a Luer Lock to ensure perfect locking and sealing on the syringe during use.

The lateral orifice at the end of the cannula makes it possible to direct the injection of the filling product.

The cannulas described in these instructions relate to the following models: MI, MIKR, MIKE, MIKS, MIKRM, SP, XTW, KIT 1 to 29.

Intended use

Injection of a filler (synthetic or autologous fat) under the skin for aesthetic body contouring.

Target population

Male or female, young adult or adult patient, including people who require an alternative way of surgery.

Not to be used on children or pregnant or lactating women.

Intended users

The micro cannula is to be used by physicians or health care professionals trained to the injection of fillers.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Contra-indications

⚠ Do not use in the blood system: only for subcutaneous injection

⚠ Do not reuse: Re-use of the device may lead to incorrect flow, increased risk of infection or other malfunctions of the device.

⚠ Micro cannulas should not be used on children or pregnant or breastfeeding women.

⚠ Silicone injections are prohibited.

Hematologic abnormalities, radiotherapy or prior chemotherapy, chronic drug use, drug use, diabetes mellitus, connective tissue diseases

Other contraindications are: pregnancy, bleeding disorders; immune deficiency; heart, liver and kidney failure; allergies to local anesthetics; pacemaker and severe heart rhythm disorders; psychiatric disorders; unstable motivations; patients with severe systemic disease, cardiovascular disease, severe coagulation disorders, including thrombophilia, embolism or thrombophlebitis, infectious diseases, wound healing.

Possible side effects

Injection site reactions (Erythema, Edema, Pain/tenderness, Bruising/hematoma, Itching, Nodule/abscess, Systemic responses, Biofilm, Inflammation, Osteophyte)

Infection (Erythema, Edema, Pain/tenderness, Acne papule formation, Nodule/abscess, Herpes outbreak, Biofilm, Herpes outbreak, Foreign-body granuloma, cellulitis)

Hypersensitivity (Erythema, Edema, Pain/tenderness, Non-fluctuant nodules, Migration of filler material)

Technical and placement errors (Bumps/lumps, Asymmetries, Contour irregularities, Compromised muscle function, Dysesthesias, paresthesias, and anesthesia, Immune reactions, Aphasia and hemiparesis)

Skin discoloration (Redness, Whiteness, Hyperpigmentation, Persistent discoloration, Persistent scarring)

Vascular compromise (Blurred vision, Loss of vision, Pain, Blanching, Tissue necrosis, Vascular perforation, blood loss)

Others (seroma formation, fatty embolism, external genital swelling, pneumothorax, pulmonary embolism, nerve injury, syncope (Vasovagal origin), diffuse sensitivity, induration, panniculitis, fat necrosis, hemorrhagic necrosis of fat, non-specific inflammation)

Instructions for Use

To attach the cannula to the syringe, firmly grasp the cannula with its plastic protector and:

- For a simple male luer syringe: push the luer of the syringe as far as possible into the hub of the cannula.

- For a syringe equipped with a male Luer Lock: firmly push the cannula into the end of the syringe by turning it clockwise.

To remove it, grasp the cannula with one hand while turning the syringe counterclockwise with your other hand. Carefully remove the cannula.

1. Sting the skin

The diameter of the pre-hole needle should be the same as or slightly greater than the diameter of the micro cannula used.

If you use our kits, the needle included in the kit is adapted to the diameter of the cannula.

⚠ Do not use the pre-hole needle to inject the filler. The use of a needle increases the risk of vascular injection and diffusion of emboli in the terminal capillary circulation.

! Do not recap the needle

2. Insert the cannula

Insert the cannula into the incision previously made.

Pinch the skin to create a small drop of blood in the punch hole that should help you locate the hole.

Stretch the skin in the opposite direction with the hand that does not inject to help enlarge the perforation hole.

Place the index finger of the injection hand near the end of the micro cannula to help guide the cannula into the skin, or hold the syringe like a dart.

3. Slide the micro cannula into the skin

As you enter the skin, rotate the cannula tangentially to the surface of the skin using the flexibility of the cannula.

To allow it to slide, it is necessary to stretch the skin in the opposite direction of the progression of the cannula

In case of doubt and before injecting the product, the practitioner "must carry out a short aspiration" which will confirm to him that he has not stuck in a vein.

4. Inject the filling product

Start injecting while doing back and forth movements along the entire length of the micro cannula. To change direction, remove the cannula almost completely from the skin, return to the second step (leaving the tip at the desired depth).

Inject small quantities of products, applying low pressure.

5. Massage

It is recommended to massage the skin after injection to ensure that the product has been evenly distributed.

! Do not massage in case of skin bleaching

Check the integrity of the cannula when removing it.

Warnings - Cautions

This product is device that may be contaminated after use. Handle and dispose of with other contaminated devices according to established medical protocols and other applicable regulations.

Intended for single use - Do not re-sterilize

Store in a clean and dry place at room temperature (5-30 ° C). Keep away from moisture (30-60%) and sunlight. Store in the original packaging

Read the following instructions carefully before use. Improper use of the instrument may cause a serious injury.

Use by date - Use this product before the expiry date which is stated on the packaging.

Check the integrity of the protective primary sterility packaging. NEVER use a device whose primary packaging is damaged.

Micro-cannulas should be handled and operated by healthcare professionals completely familiar with their use, assembly, and disassembly. Use instruments for their intended surgical purposes only.

⚠ Silicone injections as a filler are a dangerous and unacceptable option.

Check that the product you are injecting complies with the regulations. Several products are prohibited from use.

⚠ Do not use these cannulas for fat injection when the diameter is less than 21G.

⚠ The risk of vascular complications associated with facial aesthetic procedures has been addressed previously in the literature. It concerns injury to ophthalmic and retinal arteries during the injection of facial fillers resulting in irreversible serious adverse events. Physicians performing facial filler injections should have a proficient knowledge of anatomy.

Adapt the diameter of the cannula to the viscosity of the injected product.

Be sure to apply good aseptic practices.

Check the integrity of the cannula before use. Do not exert excessive forces that may rupture the cannula

If the cannula is broken or there is a leak, the cannula needs to be changed.

Perform the injection in a clean, calm, uncluttered and illuminated environment.

Results of this procedure will vary depending upon patient age, surgical site, and experience of the physician.

Results of this procedure may or may not be permanent.

The cannulas do not contain latex, PVC or phthalates. The materials used are not of animal or biological origin.

Box of 20 pieces.

510(k) labelling

INSTRUCTIONS FOR USE

CANNULAE AND NEEDLES

WARNINGS:

• Aesthetic Group instruments are to be used in accordance with these instructions for use. Read all sections of this insert prior to use. Improper use of the instrument may cause a serious injury.

• Aesthetic Group instruments should be handled and operated by healthcare professionals completely familiar with their use, assembly, and disassembly. Use instruments for their intended surgical purposes only.

• The Aesthetic Group cannulae and needles are packaged sterile and are intended for single use only. Re-use could result in infection or injury to the patient.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

INDICATIONS FOR USE:

The aspiration and infusion cannulae and needles are indicated for aesthetic body contouring and general tissue aspiration.

PACKAGED STERILE

INTENDED FOR SINGLE USE

WARNINGS

a. This device will not, in and of itself, produce a significant weight reduction

b. This device should be used with extreme caution in patients with chronic medical conditions, such as diabetes, heart, lung, or circulatory system disease or obesity.

c. The volume of blood loss and endogenous body fluid loss may adversely affect intra and/or postoperative hemodynamic stability and patient safety. The capacity of providing adequate, timely replacement is essential for patient safety.

PRECAUTIONS

d. This device is designed to remove localized deposits of excess fat through small incisions and subsequently transfer the tissue back to the patients.

e. Use of this device is limited to those physicians who, by means of formal professional training or sanctioned continuing medical education (including supervised operative experience), have attained proficiency in suction Lipoplasty and tissue transfer.

f. Results of this procedure will vary depending upon patient age, surgical site, and experience of the physician.

g. Results of this procedure may or may not be permanent.

h. The amount of fat removed should be limited to that necessary to achieve a desired cosmetic effect.

i. If harvested fat is to be reimplanted, the harvested fat is only to be used without any additional manipulation.

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